

SureLock[®]

All - Suture Anchor System


Surgical Technique Guide for Bankart Repair

Locking technology solves the mystery of anchor displacement.

Take a closer look...



SureLock[®]
All-Suture Anchor

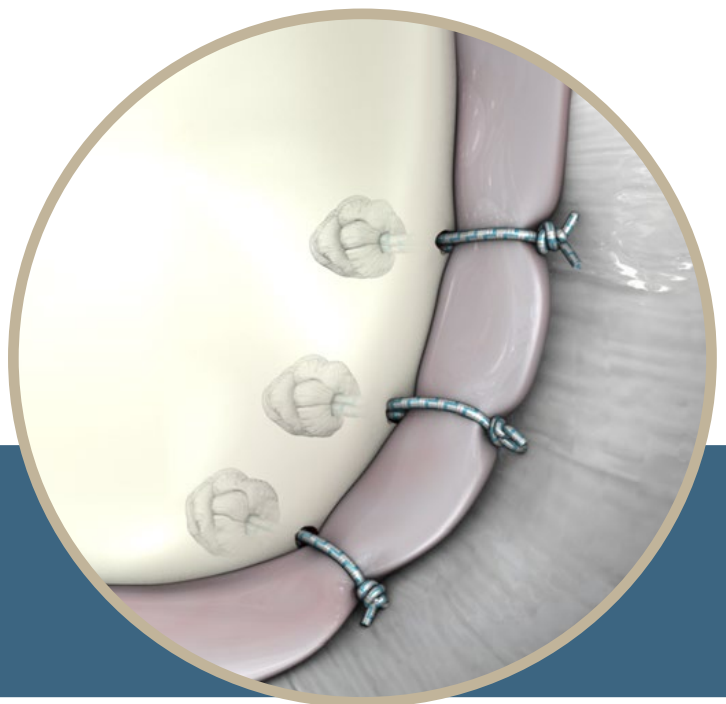


SureLock® All-Suture Anchor System

The Cayenne Medical, Inc. SureLock® All-Suture Anchors are intended to provide soft tissue fixation to bone. The non-absorbable SureLock® All-Suture Anchor is attached to an inserter and available in 1.4mm and 2.2mm sizes.

Site Preparation

- 1 Prepare the repair site with a shaver/burr/rasp. This will help promote tendon-to-bone healing.
- 2 Place the SureLock® obturator into the SureLock® drill guide and insert through the anteroinferior portal to position the tip on the glenoid surface. Establish appropriate alignment and position of the implantation site.


Note: The anterosuperior and anteroinferior portals are shown.

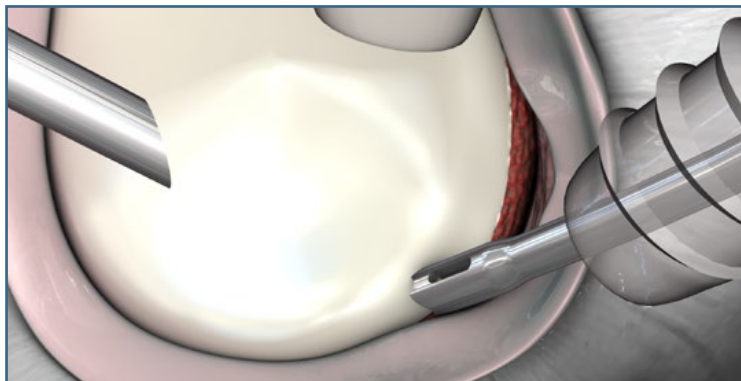


** Curved and straight drill guides are available for easy access to all tear sites.*

- 3 Remove the obturator and insert the SureLock® drill into the proximal end of the drill guide.


Note: The SureLock® Drill Guide must be used when drilling the pilot hole.

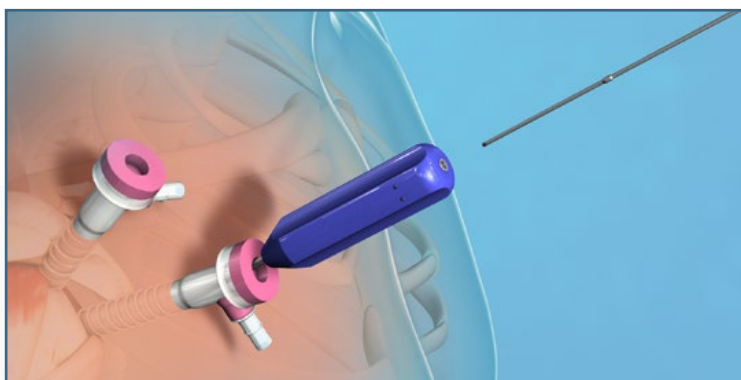

- 4 While holding the drill guide steady on the glenoid, create a pilot hole by drilling until the stop collar on the drill contacts the proximal end of the drill guide. A positive stop will indicate proper depth has been reached.

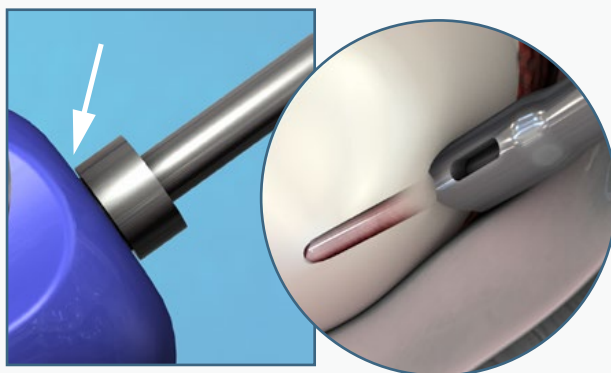

- 5 Remove the drill while maintaining alignment and position of the drill guide on the bone surface. This will allow for easy insertion of the SureLock® All-Suture Anchor.

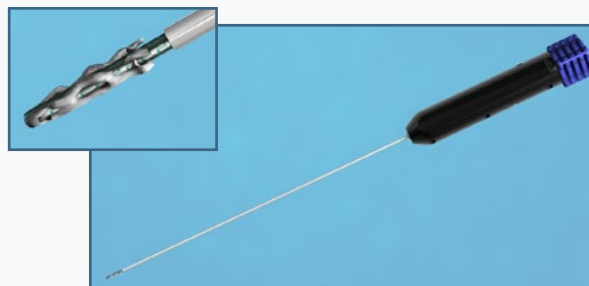


SureLock® All-Suture Anchor with Inserter

Deployment

- 6 Insert the SureLock® anchor into the proximal end of the drill guide. Lightly mallet the anchor into the pilot hole until the inserter handle comes into contact with the proximal end of the drill guide.


Note: The SureLock® Drill Guide must be used for proper deployment and fixation of the SureLock® All-Suture Anchor.

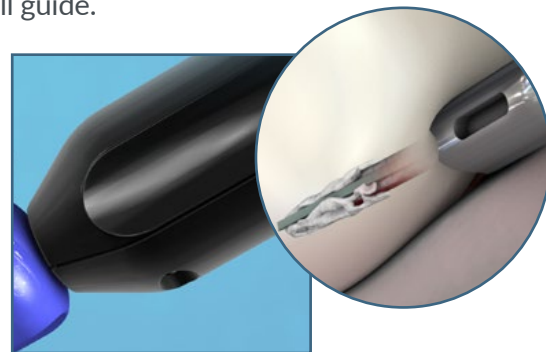

Note: The SureLock® Anchor is fully seated when the inserter handle meets the drill guide.

- 7 Turn the deployment knob on the proximal end of the inserter clockwise until it stops.


** Inserter-controlled delivery method eliminates manual tensioning and consistently deploys the anchor beneath the cortex.*

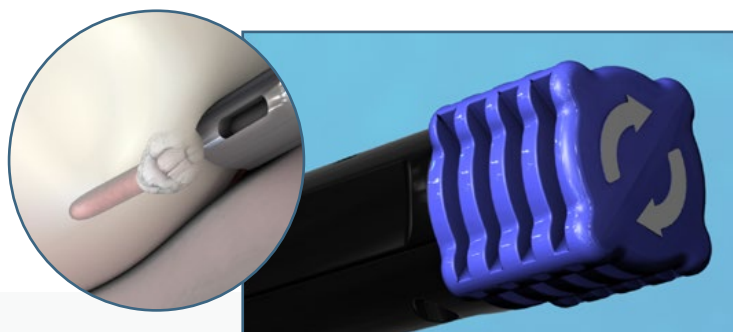



Final Repair




- 8 Pull back slowly on the inserter handle to release the inserter from the anchor. Discard the inserter handle. Remove the drill guide.

** High Strength UHMWPE Suture allows for smooth, easy suture sliding and knot tying.*

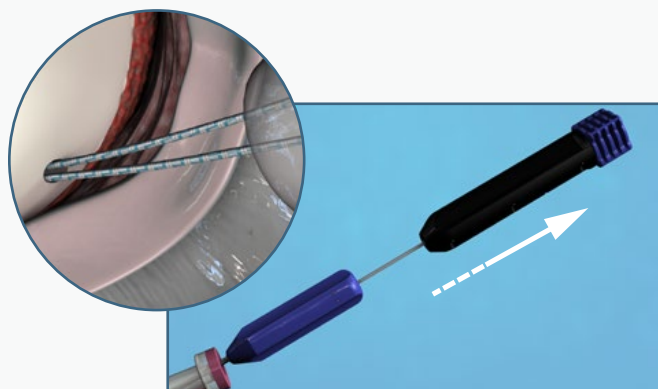

- 9 Pass suture through tissue and reattach the tissue as appropriate.

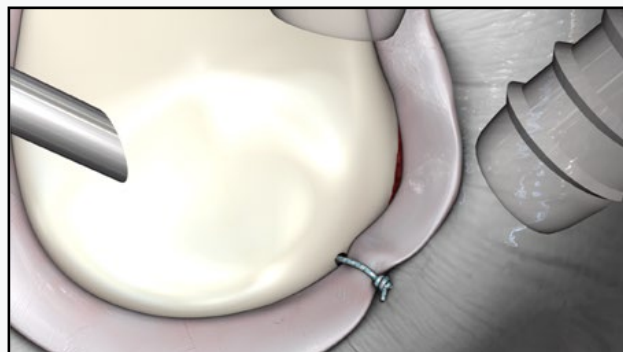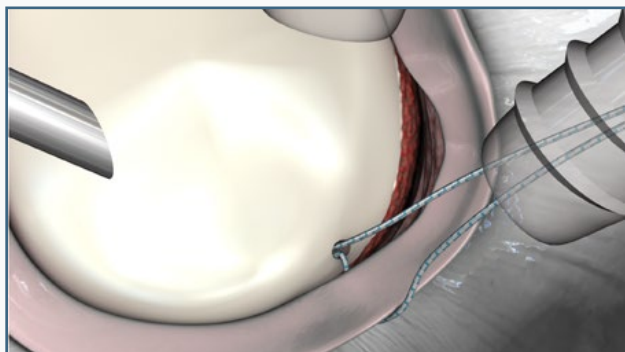

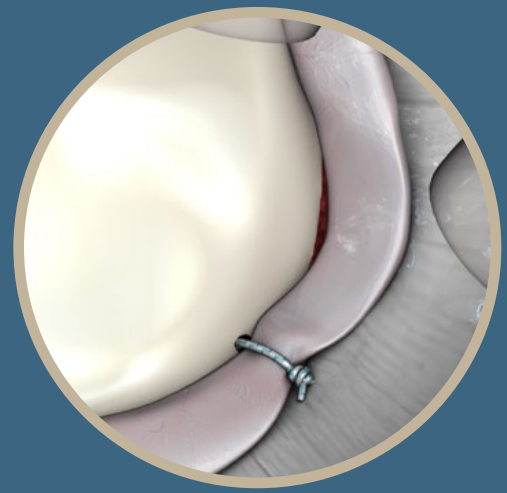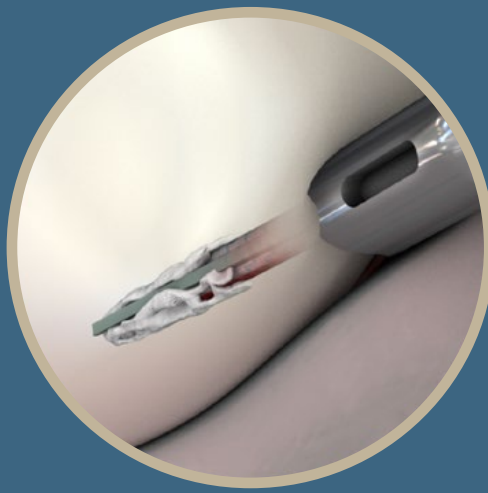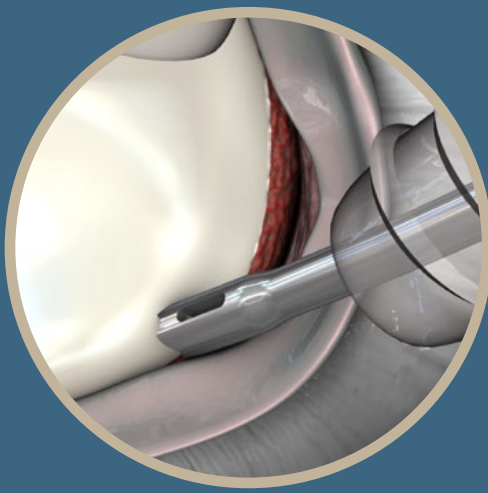

SureLock® All-Suture Anchors (*Sterile*)

CM-9614F	SureLock® All-Suture Anchor, 1.4mm pre-loaded w/(1) Size 2 UHMWPE Suture, Flexible Insertor
CM-9622F	SureLock® All-Suture Anchor, 2.2mm pre-loaded w/(2) Size 2 UHMWPE Suture, Flexible Insertor

SureLock® Disposable Instruments (*Sterile*)

CM-9620	Drill, for 2.2mm SureLock® All-Suture Anchor
CM-9620S	Drill, with Centering Sleeve, for 2.2mm SureLock® All-Suture Anchor
CM-9640	Drill, for 1.4mm SureLock® All-Suture Anchor
CM-9640S	Drill, with Centering Sleeve, for 1.4mm SureLock® All-Suture Anchor

SureLock® Instruments (*Non-Sterile*)

CM-9621	Obturator, for 2.2mm SureLock® All-Suture Anchor
CM-9623	Drill Guide (straight), for 2.2mm SureLock® All-Suture Anchor
CM-9625	Drill Guide (curved 25°), for 2.2mm SureLock® All-Suture Anchor
CM-9641	Obturator, for 1.4mm SureLock® All-Suture Anchor
CM-9643	Drill Guide (straight), for 1.4mm SureLock® All-Suture Anchor
CM-9645	Drill Guide (curved 25°), for 1.4mm SureLock® All-Suture Anchor

online at cayennemedical.com | 1-888-229-3661



Contact Cayenne Medical for order information